WALKERS

W100
W100H
W100R
W100RH
W101
W201

Instrucciones de adaptación · تعليمات المستخدم · Pokyny K Nastavení · Anleitung
Tilpasning instruktion · Οδηγίες εφαρμογής · Kohandamisjuhised · Fitting instructions · Sovitusohjeet
· Instructions d'adaptation · Upute za postavljanje · Rögzítési utasítások
Istruzioni per l'adattamento · Uždėjimo instrukcijos · Pielāgošanas instrukcijas · Montage-instructies
Tilpassingsinstruksjon · Instrukcja obsługi · Instruções de adaptação · Instructiuni de reglare
Инструкции по адаптации · Návod na použitie · Utprovningsinstruktion

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Description and intended use:

Upper ankle joint immobilising orthosis. Full and comfortable immobilisation of the upper ankle joint including the foot, ankle and calf, enabling unlimited walking thanks to its rocker sole. Designed to provide stability, support and compression to the ankle. Easy to fit. Stabilises the foot/ankle for complete control during the healing period, and, combined with the rocker sole, it makes walking easier, especially in the exit phase. Breathable, washable, latex-free lining. Non-slip sole.

Indications:

Situations where a high ankle immobilisation is required as: acute ankle sprains, lesions on the soft calf tissues, fractures caused by stress to tibia and fibula, stable fractures of foot and ankle, stable distal fractures of tibia and fibula, surgery of Achilles tendon. This product has been designed as a complement to the variety of common medical treatments for the mentioned pathologies. The prescription and duration of the treatment will be always determined by the doctor.

Contraindications:

The use of this product is recommended only when the fracture is proven to be stable and has acceptable limits of angular and rotational deformity.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe the orthosis and determine the duration of treatment.
- The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
- The use of a sock is recommended to reduce direct contact between the product and the patient's skin.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment with the orthosis should be regularly monitored by your doctor.
- If any side effects are experienced, your doctor should immediately be informed.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components (straps, Velcros, fabric, etc.) must be regularly checked to make sure they are in good condition.
- Regularly check the condition of the soles and the joints (in the ROM version). The pieces must be firmly housed in the positions determined by the prescriber. Check that they are not folded, damaged, etc. Be extremely careful when walking on wet, icy, stony, polished or other slippery surfaces. Not recommended for patients who weigh more than 120 kg.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath, shower, etc.

Instructions for proper fitting:

For all models, the fabric liner must first be placed around the leg. Remove the fabric liner from the boot walker and fasten it to the leg using the Velcro straps so that it fits snugly and comfortably. The W101 and W201 models have additional condyle pads that can be positioned on the malleoli inside the liner to provide cushioning in that area.

- Make sure that the heel is well positioned against the back of the fabric liner.
- Open the straps and slide the leg into the walker.

- Make sure that the heel and bilateral uprights are centred and start closing the straps from the toes to the ankle.
- Remove the Velcro Protectors from the bilateral uprights to secure the fabric liner and adjust the remaining straps from bottom to top.

For the W101-W201 models

These models are fitted with protective front and rear flaps. Before fastening the straps, tighten the flaps over the fabric liner to protect the leg.

Joint adjustment on ROM models (W100R, W100RH)

Warning: You must use the same adjustments on both joints. The joint must be adjusted before the limb is placed into the walker.

- Remove the locking screw using the Allen key provided.
- Rotate the joint cover to expose the pin that adjusts the bending angle. Remove the pin, select the correct setting and refasten.
- Repeat the same procedure with the extension adjustment pin.
- Rotate the joint cover until the slot is vertical and secure it by screwing in the locking screw.
- Repeat the same procedure for the joint on the opposite side.

Fabric liner adjustment on AIRE models (W100H and W100RH):

Warning: The fabric liner should only be inflated when it is correctly positioned around the injured limb and properly attached to the rigid part of the Walker. It must always be deflated whenever putting it on and taking it off.

In AIRE (H) models the fabric liner is inflatable, which increases the immobilisation of the limb.

INFLATED: Make sure the valve (black button) is closed (turned clockwise). Press the blue inflation knob built into the liner.

DEFLATED: Open the valve by turning it anticlockwise, and then press down on the inflated areas of the fabric liner to expel the air.

Positioning of the toe protector (all models):

- Once the fabric liner of the Walker is closed and secured, it is possible to attach a toe protector. The Velcro straps in the toe area must be loosened (Fig. 1).
- Slightly lift the fabric liner of the Walker in the toe area and insert a part of the toe protector underneath the fabric, making sure that the Velcro is stuck to the fabric liner (Fig. 2).
- Once secured, fold it over your fingers and stick the Velcro to the top of the fabric liner (Fig. 3 and 4).
- Once the protector is in place, close the straps again.

Product maintenance - washing instructions:

The fabric liner can be washed at 40 °C with mild soap and then air-dried.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A.