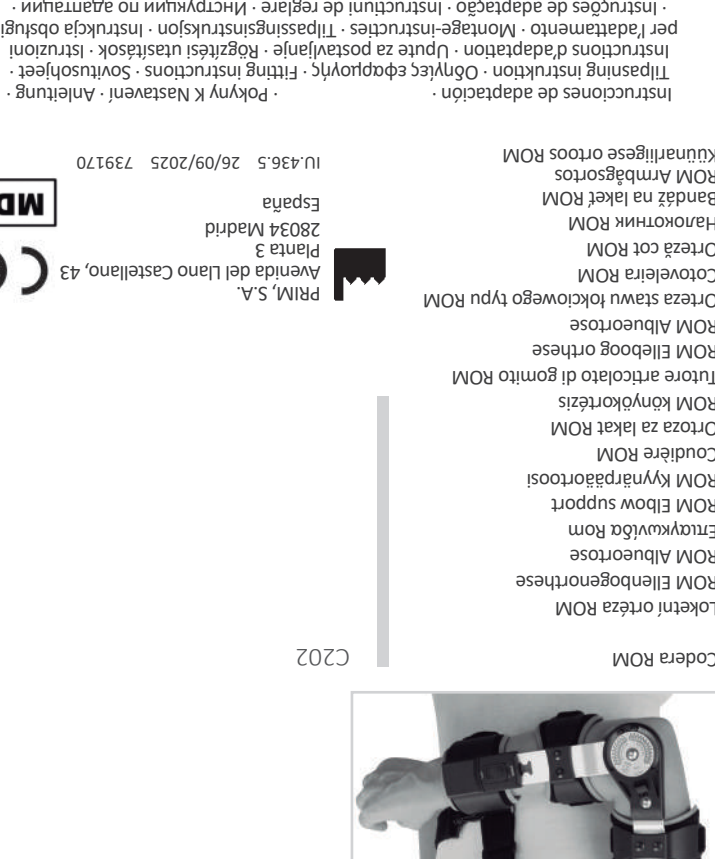
ET

Kasutusjuhised:

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ENG

Indications:

Immobilisation and adjustment of the upper limb even when control, or blocking, of the range of flexion and extension is required during the treatment of ligament injuries or sta-ble fractures. Post-operative. Disorders where flexo-extension control is required.

Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your physician or the shop where you purchased it.
- Only a physician is qualified to prescribe and determine the duration of treatment.
- The orthosis requires assembly and prior adaptation by an orthopaedic technician or qualified personnel. After the orthosis has been adapted to a patient, it should only be modified by qualified personnel.
- The orthosis is for single patient use only given it must be specifically adapted to the said patient. Do not reuse on other patients.
- For maximum effectiveness, you should respect the daily duration of use and follow the instructions of your physician or orthopaedic technician.
- To ensure a full therapeutic effect, the product must be correctly adapted.
- This orthosis must not be used for any other purpose than that prescribed. Do not let children play with this device.
- Monitoring must be frequently revised by the physician. If any secondary effects are noticed, inform the physician immediately. You must inform your physician if you suffer from any skin conditions. Avoid using topical creams during treatment.
- For hygiene and comfort it is recommended to wear a cotton garment, preferably seamless, underneath the orthosis.
- The fabric parts of the orthosis are easily removed allowing them to be washed. Clean the product regularly and maintain proper personal hygiene.
- In pressure bearing areas, the skin must not be damaged or hypersensitive. Do not wear the orthosis while having a bath, shower, etc.
- Regularly check that the straps are tense and in good condition (free of grease, cuts, tears, etc.). It is advisable to keep the Velcro fasteners free from fibres, fluff, etc., so as to maintain their fastening properties.
- The correct usage of an orthosis depends on the state of its components, thus it should be checked regularly. The sanitary personnel who monitor the patient's treatment can inform the latter of the suitability of the product or whether it should be replaced, if the product has deteriorated or worn out.

The orthosis can only be used if the locks are correctly positioned. Likewise, the tele-scope mechanism should be equally configured on both sides and properly locked. If any of the orthosis' blocking mechanisms should be faulty, consult immediately with your orthopaedic technician or the establishment where it was acquired.

Adaptation instructions

Length (telescopic device)

1. To obtain the proper length of the orthosis loosen all the straps and hold the hinge up to the external epicondyle (external part of the elbow). To adjust the length of the orthosis press the locks to release them, then lengthen or shorten as required.

- The lower part of the orthosis should reach the wrist while the upper part should be as close to the armpit as possible without causing irritation.
- Engage the lock by pressing it again. Tighten the straps in the order indicated in the illustration.

- If the straps require shortening, remove the distal Velcro first, trim, then return it to its position. The lateral struts must follow the contour of the arm.

Limit of flexion and extension (articulation)

- Press toward "UNLOCK" to unlock the hinge. Move the tabs to the required positions (extension/flexion) according to the angles indicated by the prescriber.
- Press toward "LOCK" to lock the hinge.

Maintenance of the brace - washing instructions:

Clean the hard surfaces with a damp cloth. Hand washable. Do not use bleach. Do not tumble dry. Do not iron. Do not dry care. Careful cleaning of the velcro is recommended, to eliminate threads, lint, etc., and thereby preserve correct adhesive properties.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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