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ES

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EN

Tobilas. Figure “8” elastic ankle support

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(ENG) Description and intended use: The P365 ankle support is designed to limit ankle movement in patients with mild instability characterised by a slight loss of control or stability during movement, but no significant structural damage to the ligaments. The support is manufactured in an elastic fabric that conforms to the shape of the ankle, ensuring a comfortable and effective fit. It features male and female Velcro fastening systems at each end, which help position and secure the orthosis correctly, thereby guaranteeing optimal support during use.

Indications: Mild instability of the ankle joint, characterised by a slight loss of control or stability during movement, with no significant structural damage to the ligaments.

Contraindications:

- Do not use on open fractures.
- Do not use if the area where the orthosis is placed is infected.
- Do not use on patients with vascular disorders.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the supplier you purchased it from before use.
- The initial fitting and adjustment of the brace must be carried out by qualified personnel (doctor, orthopaedic technician etc.). Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring. Always follow these general instructions for use and any specific guidelines indicated by your doctor.
- Only use in low footwear that does not interfere with the main part of the orthosis when in use. Avoid wearing boots and ankle boots.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately. Discontinue use until assessed by a specialist.
- The skin should not be damaged or hypersensitive in the areas where pressure is applied.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- For the orthosis to function correctly, its components must be regularly checked to ensure they are in good condition. The medical personnel responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- Clean the product regularly and maintain adequate personal hygiene. Care should be taken to clean the Velcro quick-release fasteners to maintain their functionality, removing any materials that may have adhered to them.
- Do not use in environments with excessive heat, fire or radiation.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this product.
- Avoid using topical creams that come into contact with the orthosis during treatment.
- The Velcro should not be fastened to the fabric of the support as this could cause fraying and deterioration. The Velcro should be fastened to the surface of the strap, which is specially designed for this purpose.

Fitting instructions:

1. Place the elastic strap on a flat surface and rest the sole of your foot in its centre (Fig. 1)
2. Bring both ends of the strap upwards, placing them on either side of your foot (Fig. 2)
3. Cross both ends of the strap over the instep of your foot, making sure it is snug but not excessively tight (see Fig. 3)
4. Wrap the ends of the strap around your ankle and fasten the Velcro closure, ensuring that the orthosis is secure and comfortable (see Fig. 4)

Fig. 1

Fig. 2

Fig. 3

Fig. 4

Maintenance instructions: Close all the Velcros before washing. Wash at a maximum of 40°C using a mild soap. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean. Do not expose to direct heat sources such as stoves, heaters, radiators or direct sunlight.

Note: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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