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ENG- Instructions of Use for the Frejka Orthosis

Read carefully these instructions before using this orthosis

Indications

Sub-luxation and congenital hip luxation. Hip dysplasia.

Fitting instructions

Place on the baby the plastic orthosis (Fig. 1) as a diaper, with legs in abduction. Place the pant (Fig. 2) with straps along the baby back. Cover the plastic orthosis with the pant making the lower Velcros of the plastic orthosis coincide with the Velcros of the pant for a perfect adjustment. For the final adjustment, close the back straps into the front quick-closing buckles. If they were too long, they can be tightened to the adequate height. Finally, proceed to close the 4 lateral straps closures, adjusting them to the width of the baby.

Cautions

The doctor/healthcare professional is the person qualified to prescribe this orthosis. The first fitting of this brace should be done by your doctor or by a technical orthopaedist. The doctor/healthcare professional is the qualified person to decide the duration of the treatment. You must follow your doctor instructions regarding the daily time of use. For a correct function, you should use the correct size of the orthosis. Do not press the orthosis too much. Take it off immediately if you observe a change of the skin colour or other problem relating to blood circulation or skin irritation. Do not exercise pressure on the articulated zones. For any doubt, consult always your doctor or the healthcare professional who fitted the orthosis. Inform your doctor if you may notice any secondary effect. This orthosis should never be used for any other use different to the one prescribed. Do not allow children to play with this device. This orthosis is designed for the use on one patient. Do not reuse on multiple patients. Considering that the patient is a baby, the state of the orthosis has to be periodically revised by a responsible adult. The baby should not wear the orthosis when bathing, having a shower, etc.

Advices for use:

Skin care:

On the pressure points, the skin should not be injured nor irritated. It is very important to revise the good state of the baby's skin at least once a day. More caution should be taken on the following areas: Shoulders, Neck, Back, Breast, Abdomen, Back Part of the Knees and Legs. The use of this product with medicaments of topic use (creams, talcum, etc.) should be supervised by your doctor. Avoid the use of creams during the treatment in order to avoid the premature deterioration of the product. For hygiene and comfort reasons and in order to protect the skin, it is advisable to use a cotton garment.

Advise during the Bath:

Bath the baby as quicker as possible in order to be the minimum time without the orthosis. It is advisable to softly massage on the pressure points and in contact with the orthosis in order to relax and tone up the baby muscles.

Care instructions

It is advisable to maintain the Velcro® free of dirt preventing them from threads, fluffs, etc, in order to keep their adhesion characteristics. Close all the Velcro® straps on the brace surface before washing the product.

Check periodically that the cinch straps are tight and in good conditions (without grease, cuts, rips, etc.).

Machine washing with lukewarm water. Do not bleach. Do not iron. Do not dry clean. Do not tumble dry. Dry on flat surface. Do not hang out.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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