**Knight CampXXI
632**

**Knight CampXXI
Abdomen Péndulo
634**

Knight

**CAMP[®]
XXI**

Instrucciones de Adaptación / Fittitting Instructions / Instructions d'adaptation / Instruções de adaptação / Istruzioni per l'applicazione / Bevestigingsinstructies / Anweisungen zur Anpassung

Mod.: 632

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Release all Velcro® straps. Place the brace at the required height and without applying tension, adjust it correctly and close the central Velcro® strap.

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Release all Velcro® straps. Put on the brace at the required height placing the abdomen into its floating from posicion and without applying tension, adjust it correctly and close the central Velcro® strap.

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Pull with the same tension the two lower straps from the rear to the front.

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Close the right lower strap over the Velcro® loop of the central closure and the left one over the right one.

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Repeat the same procedure with the upper straps.

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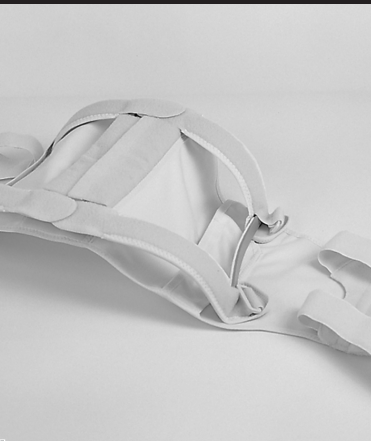
Open the Velcro® straps of the lateral parts and take the frame out of the textile. Unscrew the lateral bars in order to be able to remove the paddings from the upper and lower ring.

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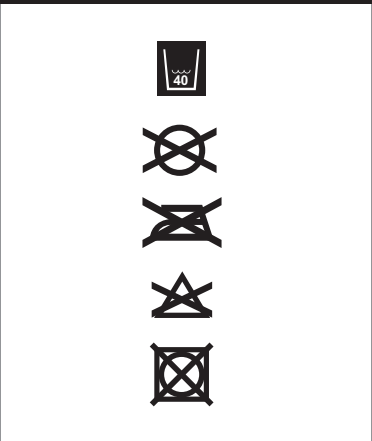
Machine wash with luke-warm water. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on flat surface. Do not hang-dry. We recommend to use the wickerwork bag supplied with the product to protect the garment during the washing. Close all the Velcro® straps before washing.

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Assemble the paddings of the upper and lower ring. Screw on the lateral bars. Assemble the textile and the frame and close the Velcro® straps of the lateral part.

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PRIM, S.A.