**Taylor CampXXI
637**

**Taylor CampXXI
Abdomen Péndulo
638**

Taylor

CAMP[®] XXI

Instrucciones de Adaptación / Fittitting Instructions / Instructions d'adaptation / Instruções de adaptação / Istruzioni per l'applicazione / Bevestigingsinstructies / Anweisungen zur Anpassung

Mod.: 637

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Release all Velcro® straps. Place the brace at the required height and without applying tension, adjust it correctly and close the central Velcro® strap.

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Release all Velcro® straps. Put on the brace at the required height placing the abdomen into a comfortable position and without applying tension, adjust it correctly and close the central Velcro® strap.

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Pull with the same tension the two lower straps from the rear to the front. Close the lower strap over the Velcro® loop of the central closure and the left strap on the right one. Repeat the same procedure with the upper straps.

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Pull the straps over the shoulders, applying just enough tension in order to give support to the back part of the brace. Cross the straps in the back.

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Pull the straps from the rear to the front with the same tension. Close the right suspender over the straps and superpose the left suspender over the right one.

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Open the Velcro® straps of the rear part and take the frame out of the textile.

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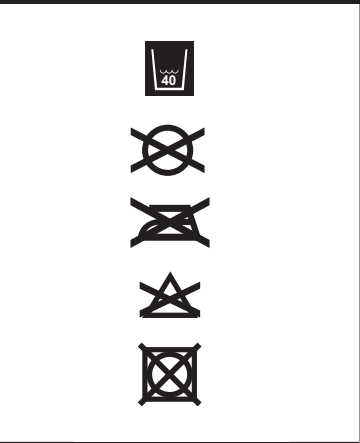
Machine wash with luke-warm water. Do not bleach. Do not iron. Do not dry-clean. Do not tumble dry. Dry on flat surface. Do not hang-dry. We recommend to use the wickerwork bag supplied with the product to protect the garment during the washing. Close all the Velcro® straps before washing.

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Assemble the textile and the frame and close the Velcro® straps of the rear part.

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Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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A qualified Medical Practitioner is the only person who can prescribe and decide the duration of treatment. The initial fitting should be done by, or under the supervision of an orthotist. Follow the instructions specified by your Medical Practitioner regarding wearing times to achieve maximum results. For hygienic and comfort reasons, you may want to wear a cotton garment underneath your back support. Periodically check the condition of the straps to ensure they are free from grease or damage. A qualified medical practitioner should supervise the use of topical drugs.

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Prim S.A.